

WASTE MANAGEMENT

- THE IMPORTANCE OF SAFE AND EASY COLLECTION OF THE USED SYRINGES AND NEEDLES

POLYSAFE® Safety Boxes are constructed for this purpose – to prevent both health care workers as well as communities from being exposed to unnecessary risks from contaminated syringes and needles.

Due to the fact that the safety boxes are flat packed, easy to carry and assemble and that they are made from recycled solid board, the POLYSAFE® Safety Boxes are already the solution in many field hospitals, health clinics, for many NGO's and UN organizations- all over the world.

ON SITE DESTRUCTION

Under extreme primitive conditions and where it is important to get the used syringes out of circulation as fast as possible, we recommend using **POLYSAFE® 5L Safety Box and Incineration Container**.



One POLYSAFE® Safety Box 5L takes over 100 used 0,5 ml syringes with needles and caps



Easy to ignite the POLYSAFE® Safety Box



After 2 minutes



After 9 minutes



After 20 minutes



After 30 minutes

One container can be filled with approx 150 used syringes and it can be burned on site (see pictures).

Once filled, you can burn the whole box as it is. To ignite the box, use a lighter or a match (see picture 2) (NB: no fluid required).

After approx. 30 minutes, all that is left are the needles, BUT they are no longer contaminated – due to the high heat in the burning process.

POLYNOR AS has delivered its POLYSAFE® Safety Boxes since 1994 and the boxes are approved by WHO.

We have POLYSAFE® Safety Box and Incineration Containers in stock for immediate shipping.

We can ship to all over the world.

It is important to dispose of the syringe immediately – and in a safe manner. Use POLYSAFE®.